

PARTY SAFE™

Rapid, Over-the-Counter Fentanyl test strips (5 test pack)

FAST RESULTS

- Readable results in only 5 minutes

DEPENDABLE RESULTS

- Only tests for Fentanyl
- No deciphering between substances
- Same test product used by U.S. Government agencies and Law Enforcement

EASY 3-STEP TESTING

- Quick, easy-to-read reference instructions provided

RISK REDUCTION

- May help to reduce the risk of overdose and harmful effects

MADE IN THE USA

HOW DO I RECOGNIZE AN OVERDOSE?

- Small, constricted “pinpoint pupils”
- Falling asleep or loss of consciousness. Slow, shallow breathing
- Choking or gurgling sounds
- Limp body
- Pale, blue, or cold skin

FENTANYL FACTS

Fentanyl is a potent synthetic opioid drug approved by the Food and Drug Administration for use as an analgesic (pain relief) and anesthetic.

It is approximately 100 times more potent than morphine and 50 times more potent than heroin as an analgesic. Under the United States Controlled Substances Act of 1970 Fentanyl is a Schedule II narcotic.

According to the U.S. Centers for Disease Control and Prevention U.S. deaths due to Fentanyl nearly quadrupled from 2016 to 2021.

To learn more, visit the U.S. Drug Enforcement Administration [<https://www.dea.gov/factsheets/fentanyl>] and the U.S. Centers for Disease Control and Prevention [<https://www.cdc.gov/stopoverdose/fentanyl/>].




- Because this list is not exhaustive, call 911 or seek immediate medication attention if you think you or someone else has overdosed or is at risk of overdose

INSTRUCTIONS FOR USE

STEP 1

Put a small amount of substance into a container.

Add a small amount of water (one ounce or shot) and mix.

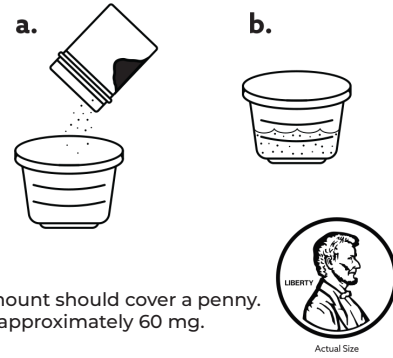

STEP 2


Dip the end of the strip for 15 seconds.

Lay the test strip flat.



STEP 3

Read results at 5 minutes.

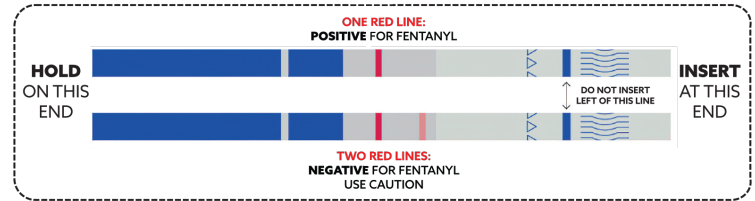

WARNINGS

- Read, understand, and follow the instructions. Your failure to do so could result in false positive or false negative test results, severe injury or death.
- Even if the results of the test(s) are negative for fentanyl, anyone consuming the tested substance could overdose, sustain severe injury, or die.
- Do not consume illicit drugs, especially if you suspect that they contain, or the test result is positive for, fentanyl.
- The test does not detect the presence of substances other than fentanyl, and it does not detect all forms of fentanyl.
- The test does not detect the percentage of fentanyl present in the tested substance.
- Exposure to even small amounts of fentanyl, or handling of fentanyl, can cause severe injury or death.
- When testing a substance, use protective equipment, such as gloves and eye protection.
- Do not use the test after expiration or if the packaging is damaged or shows other signs of tampering.
- Dispose of the test in accordance with applicable law.
- Anyone using the tests accepts responsibility for any injury, death, criminal penalty, or civil liability that occurs from consumption of a substance, whether an illicit drug or not, and regardless of whether the substance was tested, or was not tested, for fentanyl.
- Call 911 or seek immediate medical attention if you think you or someone else has overdosed or is at risk of overdose.
- If you or someone you know requires counseling or treatment for drug use, call the SAMSHA National Helpline at: 1(800)-622-HELP (4357).

NOTICE

Not for in vitro diagnostic use. Not Food and Drug Administration (FDA) cleared for this use. Please see warning labels on the packaging for more information.

AVAILABLE PACKAGING CONFIGURATIONS

• 1 Product Box (5.5"x 3.75"x 0.75")

Content: 5 Fentanyl test strips in a foil wrapper, 1 instruction card

• UPC/GTIN-13 code:



• 1 POP Display Tray (5.5"x 3.75"x 8")

Content: 12 product boxes

• Master Case

Content: 2 POP display trays
= 24 product boxes



To order contact us at: **1.800.397.0670**,
diagnostics@versea.com, or **www.verseadiagnostics.com**

